

Bleeding Control Instructions

INSTRUCTIONS ARE FOR INFORMATION PURPOSES ONLY AND ARE NOT INTENDED TO BE A SUBSTITUTE FOR PROFESSIONAL MEDICAL TRAINING

ENSURE PERSONAL SAFETY PRIOR TO PROVIDING CARE


APPLY PROTECTIVE GLOVES

- TO PROTECT FROM BLOODBORNE PATHOGENS, APPLY GLOVES PRIOR TO CONTACT WITH BLOOD



C-A-T® GEN7 TOURNIQUET APPLICATION

- USED TO CONTROL MAJOR BLEEDING FROM ARMS AND LEGS – **APPROPRIATE AREAS HIGHLIGHTED IN BLUE IN IMAGE**
- APPLY 2-3 INCHES ABOVE WOUND, NOT ON A JOINT OR DIRECTLY OVER INJURY
- PULL RED TAB AND SECURE STRAP FIRMLY OVER EXTREMITY, SECURING WITH VELCRO
- TIGHTEN BY TWISTING PLASTIC ROD UNTIL BLEEDING STOPS
- SECURE ROD USING CLIP – SEE IMAGE

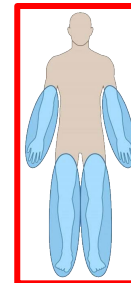


USE TOURNIQUET FOR ARM OR LEG HEMORRHAGE



ROD SECURED
IN CLIP

HEMORRHAGE CONTROL DRESSING – QuikClot®

- FOR JUNCTIONAL HEMORRHAGE – **APPROPRIATE AREAS HIGHLIGHTED IN GREEN IN IMAGE**
- PACK GAUZE INTO WOUND CHANNEL, COMPLETELY FILLING INJURY
- APPLY 3-4 MIN. OF DIRECT PRESSURE
- COVER WITH PROVIDED PRESSURE BANDAGE BY WRAPPING SNUGLY AROUND WOUND
- SECURE PRESSURE BANDAGE WITH CLIP

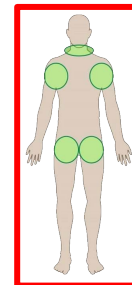

USE QuikClot & PRESSURE BANDAGE FOR NECK, SHOULDER, OR GROIN



Always Have a Con10gency™

sales@con10gency.com • (855) 590-4065 • www.con10gency.com